



DISPATCH® HOSPITAL CLEANER DISINFECTANT WITH BLEACH

DISPATCH® HOSPITAL CLEANER DISINFECTANT WITH BLEACH is a unique, stabilized sodium hypochlorite and detergent solution that kills pathogens fast—with a one minute contact time for a broad range of microorganisms including TB and MDROs (Multidrug-Resistant Organisms) such as *Acinetobacter baumannii*, *Enterobacter aerogenes*, *Enterococcus faecium* Vancomycin resistant (VRE), *Klebsiella pneumoniae*, *Staphylococcus aureus* Methicillin resistant (MRSA), and *Pseudomonas aeruginosa*. It is pre-mixed, ready-to-use, and is equivalent to the 1:10 bleach solution strength that is recommended by the CDC for effective disinfecting. This product meets OSHA guidelines for protection of healthcare workers from bloodborne pathogens.

DISPATCH® HOSPITAL CLEANER DISINFECTANT WITH BLEACH

► Fast Acting Cleaner Disinfectant

- DISPATCH® HOSPITAL CLEANER DISINFECTANT WITH BLEACH is a multipurpose, broad spectrum cleaner disinfectant for use on hard, non-porous surfaces. It may be used on stainless steel, glazed ceramic tile, hard plastics, non-porous vinyl, painted surfaces, Plexiglas, laminated plastic countertops, fiberglass (sealed) surfaces, glazed porcelain, and glass.
- May be used for instrument pre-cleaning and disinfection of non-critical instruments.
- Kills HIV, HBV, and HCV on pre-cleaned environmental surfaces and objects previously soiled with blood or body fluids. Ideal for healthcare or other settings.

► Convenient Packaging

- DISPATCH® HOSPITAL CLEANER DISINFECTANT WITH BLEACH is ready to use in convenient spray bottles. No mixing and labeling of bleach solutions required.

MULTIPLE APPLICATIONS:

Medical

- Autoclaves, blood glucose monitors, IV poles, exam tables, bed railings, dialysis machines, respiratory equipment, ultrasound transducers and probes, gurneys, mammography equipment, plastic mattress covers, phlebotomy trays, reception counters, stretchers, wash basins, and wheelchairs

Dental

- Chairs, counters, operator hard surfaces, and light lens covers

Veterinary

- Animal equipment, cages, fountains, and clinic surfaces

Laboratory

- Countertops, autoclaves, and blood glucose monitors

General Use

- Doorknobs, hand railings, changing tables, high chairs, and playpens



DISPATCH® is available through Cardinal Health. Contact Cardinal Health at 800.964.5227 or visit cardinal.com.

Order Information:

- 8 oz. Pump Spray Bottle, 15 per case 68963
- 22 oz. Trigger Spray Bottle, 8 per case 68967
- 32 oz. Trigger Spray Bottle, 6 per case 68970
- 32 oz. Pull Top Bottle, 6 per case 68832
- 64 oz. Refill Bottle, 6 per case 68973
- 128 oz. Refill Bottle, 4 per case 68978
- Spill Kit, 6 per case SK960

EPA Registration No. 56392-7



DISPATCH® HOSPITAL CLEANER DISINFECTANT WITH BLEACH

Germicidal Efficacy:

In 30 SECONDS:

- ▶ *Mycobacterium bovis* (TB)

In 1 MINUTE:

Bacteria:

- ▶ *Acinetobacter baumannii*
- ▶ *Enterobacter aerogenes*
- ▶ *Enterococcus faecium* Vancomycin resistant (VRE)
- ▶ *Escherichia coli* ESBL
- ▶ *Klebsiella pneumoniae*
- ▶ *Pseudomonas aeruginosa*
- ▶ *Salmonella enterica*
- ▶ *Staphylococcus aureus*
- ▶ *Staphylococcus aureus* Methicillin resistant (MRSA)
- ▶ *Streptococcus pyogenes*

Viruses:

- ▶ Avian Influenza A Virus
- ▶ Canine Parvovirus
- ▶ Feline Panleukopenia Virus
- ▶ Hepatitis A Virus (HAV)
- ▶ Hepatitis B Virus (HBV)
- ▶ Hepatitis C Virus (HCV)
- ▶ Herpes Simplex Virus (HSV-2)
- ▶ Human Immunodeficiency Virus Type 1 (HIV-1)
- ▶ Influenza A Virus
- ▶ Norovirus
- ▶ Poliovirus Type 1 (Mahoney)
- ▶ Rhinovirus
- ▶ Rotavirus

Fungi:

- ▶ *Trichophyton mentagrophytes* (Athlete's Foot Fungus)

▶ Meets Industry Safety Standards

OSHA has approved the use of bleach solutions as an effective means of protecting healthcare workers from the risk of contamination by bloodborne pathogens (Source: OSHA Instruction CPL 02-02-069, November 27, 2001). OSHA requires that conventionally prepared bleach solutions be made up daily (every 24 hours), and left on the surface until the bleach has air dried. Gross contamination (visible soil) must be cleaned up first with soap and water, followed by use of the bleach solution for disinfecting.

DISPATCH® HOSPITAL CLEANER DISINFECTANT WITH BLEACH eliminates the labor costs associated with mixing and labeling bleach solutions because it is pre-mixed, ready-to-use, and stable until the expiration date on the package. Unlike bleach and water solutions, DISPATCH contains detergent so it cleans and disinfects in just one step; no pre-cleaning required. DISPATCH has an appealing fragrance and also contains an anti-corrosive ingredient that makes it less damaging to surfaces or equipment than conventionally prepared bleach solutions.

▶ Expiration Dating

The expiration date should be visible on the package. If not, call customer service at 800.234.7700 for assistance or contact us at caltechindustries.com.



BIOHAZARD SPILL KITS

Designed for efficient removal of infectious body fluids, our BIOHAZARD SPILL KITS save time and simplify procedure writing. When an emergency spill occurs, kits eliminate the search for gloves, absorbents, and disinfectants. Everything needed to contain the spill, remove and dispose of spilled body fluids, and clean and disinfect the spill site is available in one handy kit.

SAFETY INFORMATION

▶ Material Safety Data Sheets

Complete MSDS information can be obtained at caltechindustries.com.

▶ Precautions

DISPATCH should be used in a ventilated area – general room ventilation is adequate for most users.



800.234.7700
caltechindustries.com

® Registered trademarks of Caltech Industries, Inc.